

MYCOFAST[®] *Revolution* Screening Method

UROGENITAL MYCOPLASMA DIAGNOSIS

NEW



Detection, Enumeration, Identification and Antimicrobial Susceptibility Testing

- Complete diagnosis in a two-phase approach
- Antibiotics compliant with new worldwide recommendations
- Cost-effective screening in 24 hours
- Optimized sample management : specimens preserved up to 72 hours before analysis
- Highly secured and flexible method
- Visual, colorimetric reading

... The Best got Better!

BENEFITS :

Complete diagnosis

STEP 1 (MYCOFAST® Screening Revolution tray) – 24/48 h

- Detection of *U. urealyticum* (U.u) and *M. hominis* (M.h)

STEP 2 (MYCOFAST® Revolution tray) – 24/48 h

- Enumeration, Identification
- Antimicrobial Susceptibility Testing

Quality diagnosis

- **SENSITIVITY**
3 levels of detection for *U. urealyticum*
- **SPECIFICITY**
 - **MYCOFAST® Revolution:**
Identification based on natural resistance to antibiotics: Lyncomycin/SXT/Erythromycin
 - **Screening Revolution:**
Specific M.h and U.u growth media.
Enhanced performance, reliable results and reduced number of erroneous results.
- **RELIABILITY**
5 antibiotics (LVX/MXF/E/CM/TE) tested at different concentrations
- **FAST RESULTS**
S.Mh = growth activator for *M. hominis* enumeration within 24 hours

Easy to use

- Ready-to-use reagents
- **NEW** Integrated growth medium
- Visual colorimetric reading
- No instrument required


Secured and flexible use

- **NEW** transport medium:
Specimens preserved up to 72 hours before analysis
- **NEW** closing system for the *Revolution* tray

Highly accurate results

- **Detection and enumeration of *U. urealyticum* at 10³, 10⁴, and ≥10⁵ CCU/mL**
Accurate U.u enumeration to enable pathological threshold detection :
≥ 10⁴ CCU/mL for urethral samples
≥ 10³ CCU/mL for urine, sperm, endotracheal samples
- **Detection and enumeration of *M. hominis* at 10⁴ CCU/mL**
≥ 10⁴ CCU/mL for cervico-vaginal samples

METHODOLOGY :

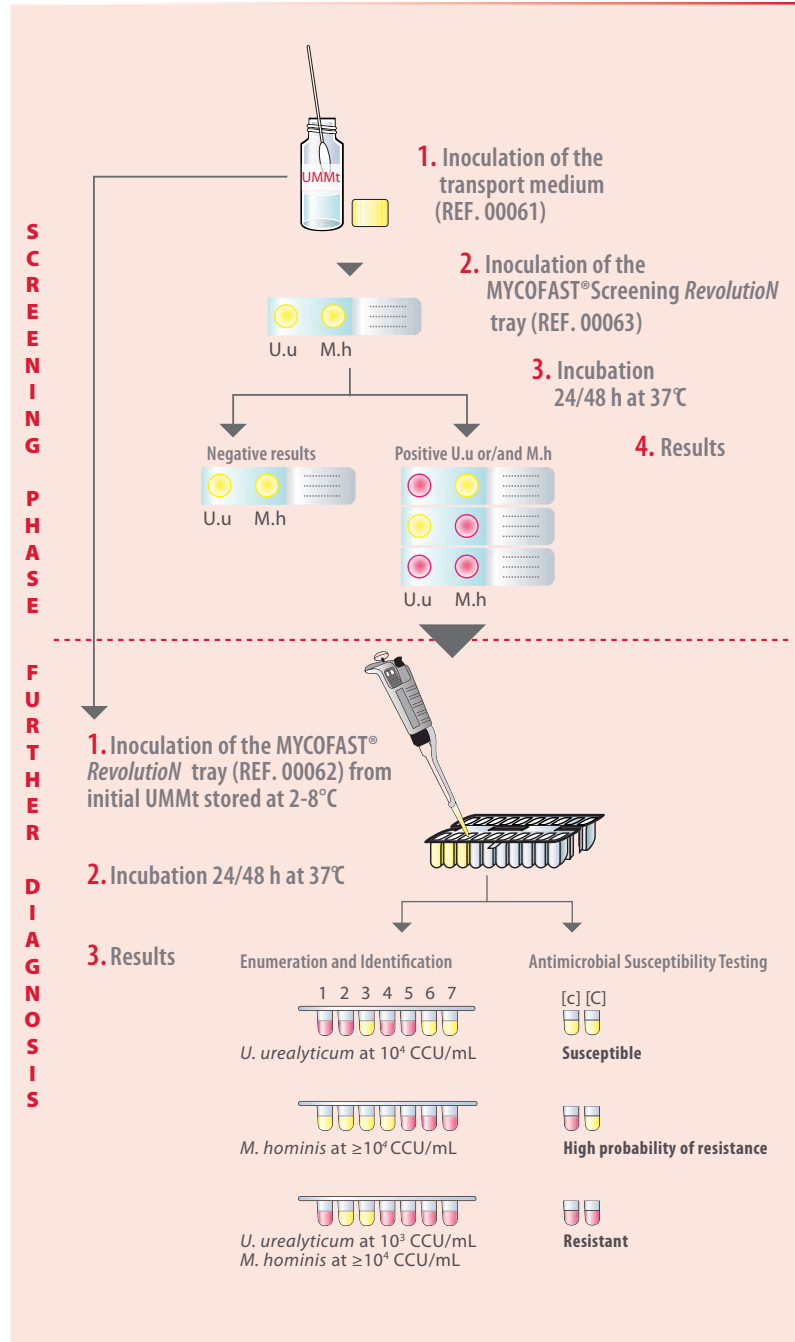

PRODUCT RANGE :

REFERENCE	NAME	QTY	PURPOSE
00061	UMMt <i>Revolution</i>	50 vials	Transport & Inoculation medium
00063	MYCOFAST® Screening <i>Revolution</i>	50 tests	Screening tray for detection of Uu/Mh
00062	COMPLETE MYCOFAST® <i>Revolution</i>	25 tests	Complete tray for further diagnosis
00064	MYCOPLASMA Stabilizer (optional)	100 tests	Additive for sample storage > 56 hours
00900	MYCOPLASMA CONTROL (optional)	12 tests	U.u and M.h control strains

Distributor Contact Information

Product not available in the US
Please contact your sales representative for product availability in your country

WORLDWIDE OFFICES:

Belgium T: +32 9 2820 531
Brazil T: +55 27 3025 1415
France T: +33 4 83 36 10 82
Italy T: +39 02 48 40 35 42

Middle East & Africa T: +971 4 375 2744
Serbia T: +381 11 2467119
Switzerland T: +41 26 663 86 60
The Netherlands T: +31 485 511 675

UK T: +44 1268 493377
United States T: +1 609 216 7360
855-ELITECH (855-354-8324)

EM T: +33 4 94 88 55 00
www.elitechgroup.com
info@elitechgroup.com

